





Sanitary installation		MEIKO	On-site
1.1	Drain TopLine	Drain trap ø 110mm	Floor sleeve DN 100
1.2	Drain stop sink	Drain trap ø 75mm	Floor sleeve DN 70
2.4	Hot water	R 1/2" Dirt collector	Inlet pipe DN 15, Stop-cocks, 45-60°. Flow: 18l/min., flow pressure: 100-500 kPa
2.5	Cold water	R 1/2" Dirt collector	Inlet pipe DN 15, Stop-cocks, 5-25°C. Flow: 18l/min., flow pressure: 100-500 kPa
Electrical installation			3 x 15A protection. Wall connection box and main fuse. Protective measure as well as the connection of the equi-potential banding must be carried out according to IEC 364-5-54 and conform to the local and national utilities regulations. Do not place smoke detector over TOPIC!
3.1	3 Gnd ~ 208/230V, 60Hz, 3,9/4,6kW ⊕ equipotential bonding (thread bolt M8) Protection class: Equipment Class 1, Pollution Degree 2, Installation Category II. Main switch. Connectin cable approx. 1,5m		
  			
 MASCHINENBAU GMBH & CO.KG. ENGLERSTR.3 77652 OFFENBURG GERMANY TEL. 0781/2030 E-MAIL: info@meiko.de			It is neither allowed to give this drawing without our permission to a third party for information or reproduction, nor may it be used for competition purposes. We reserve all rights! Attachment "Important information" to be considered!
Reference:			Type: TOPLINE20/A DT-ND/V THERMAL DISINFECTION DECENTRALIZED
Drawing-No.: M00-S12143 4		Project-No.:	Order-No.:
Scale: 1:25	drawn: 17.06.15 rm	checked: 17.06.15 rm	Customer checked: